



Heartsine samaritan® AED Trainer

#HST01

The Heartsine samaritan® PAD TRAINER provides all prompts, controls, and accessories for effective and realistic training. The samaritan TRAINER looks like and operates like the real thing, without the ability for delivery of a shock. A variety of user - selectable scenarios helps to facilitate training for a variety of emergency situations.

- 8 AHA /ERC scenarios
 - 4 to 2005 guidelines
 - 4 to 2000 guidelines
- easy to use
- pause function
- rechargeable battery
- AHA protocol updateable



#HST01

Pause button - stops the scenario temporarily to enable the trainer to offer additional explanation and support to the class without interruption. Scenario resumes where it was paused.

Rechargeable battery - can operate up to 6 hours, or the unit can be operated while plugged in. A Low Battery warning will advise users when the unit should be recharged.

Volume configurable— 4 selectable volume levels. Unit defaults to last user setting.

Scenarios are selectable as follows:

AHA/ERC 2005 - protocol is 1 shock followed by 2 minutes of cpr. A metronome and flashing prompt at 100 beats per minute helps to establish proper rate of compressions.

1. Shock ADVISED (SA) condition, followed by CPR
2. NO SHOCK ADVISED (NSA) condition, follow by CPR
3. CHECK PADS, followed by an SA condition, and CPR
4. Starts directly in “ANALYZING” mode, followed by SA & CPR, SA & CPR, and NSA & CPR

AHA/ERC 2000- Shock sequences followed by 1 minute of CPR each

5. SA, SA, and SA condition followed by CPR; then a SA & NSA condition followed by CPR
6. CHECK PADS condition, followed by SA, SA, NSA and CPR
7. Starts directly in “ANALYZING” mode, followed by SA, NSA & CPR
8. SA, NSA and CPR

Great for show - and - tell with customers interested in purchasing AEDS or an effective CPR/AED training tool.