


# THE AED FOR PROFESSIONALS

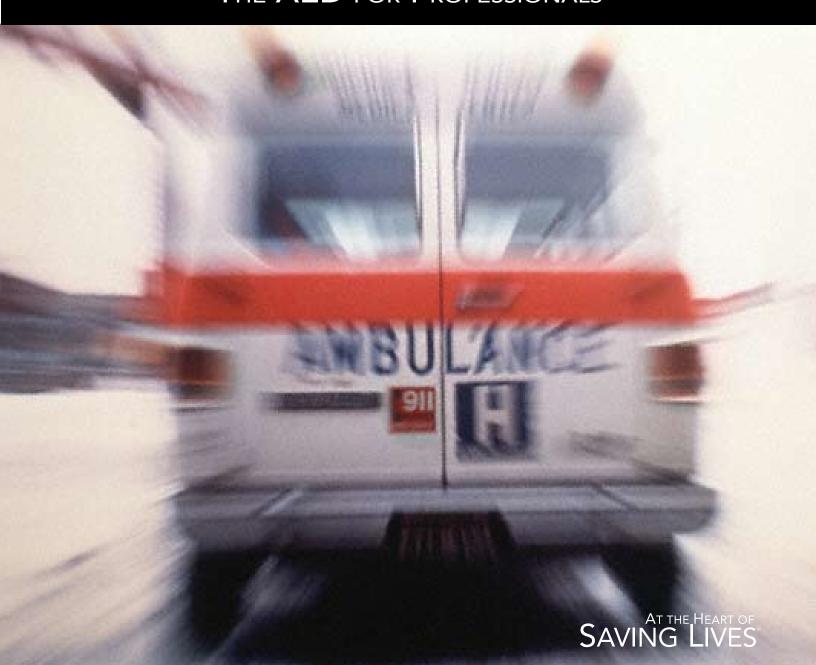

## Powerheart AED G3 Pro – Raising the Bar

The Powerheart AED G3 Pro packs more than you'd expect into an AED of this size and price point. Features include a color ECG display, 3-lead monitoring, manual override, multiple rescue capability, and the most advanced technology available to provide healthcare and rescue professionals extended capabilities for emergency treatment of sudden cardiac arrest and continuous cardiac monitoring aligned with the new AHA/ERC 2005 Guidelines protocols.

In addition to meeting the more sophisticated requirements of medical and rescue professionals, the Powerheart AED G3 Pro shares the same rescue options, programmable functions, and advanced monitoring capabilities of our flagship, feature-rich Powerheart AED G3. This includes our patented RescueReady™ technology which assures first time, every time shock delivery, as well as intuitive and comprehensive voice and text prompts that guide the user through the rescue. To learn more about the best choice in AEDs, visit our website www.powerheart.com.





9660-001 Adult Defibrillation Pads

Polarized (side-specific) Adult Defibrillation Pads. 2 year shelf life.

9131-001 A

9730-001

Adult Defibrillation Pads

Non-Polarized (non side-specific) Adult Defibrillation Pads. 2 year shelf-life.



Pediatric Defibrillation Pads

Non-Polarized, reduced energy (50 Joule) pediatric defibrillation electrode pads recommended for chest/back placement. Recommended for children less than 8 years of age or under 55 lbs. (25kg) 2 year shelf life.



5111-001 (AHA) ECC 5111-002 (IEC) For ac

**ECG Monitoring Kit** 

For accurate and continuous cardiac monitoring, provides Lead II viewpoint. Used in conjunction with the Powerheart AED G3 Pro, the ECG Monitoring Kit displays patient's non-diagnostic ECG heart rhythm on the AED display. The electrode color convention is in accordance with the AHA, IEC and AAMI specifications.



162-0108-001 IR Communications Cable

The wireless communication cable allows user to modify AED operational parameters, upload rescue data, and transfer information. The IR Communications Cable is compatible with USB ports via a standard adapter.



# POWER?HEART®

### Powerheart AED G3 Pro Accessories

Designed specifically for medical professionals, Cardiac Science's latest line of accessories offers a variety of functions that enhance the capabilities of the Powerheart AED G3 Pro. This series of accessories caters to the sophisticated demands of professional users by supporting field and clinical usage of the Powerheart AED G3 Pro, and by simplifying the data management and customization processes. Collectively or individually, Powerheart AED G3 Pro accessories enhance the professional user's capabilities.



9145-001 Pro Battery

IntelliSense Lithium Battery for Powerheart AED G3 Pro model 9300P. Warranted for 1 year or 12 hours of use, whichever comes first.



9144-001 Rechargeable Battery

A new fully-charged rechargeable battery has a capacity of delivering 60 shocks minimum, and 100 is typical. When using the Powerheart AED G3 Pro in monitoring mode, a new fully-charged rechargeable battery will monitor for 3 hours minimum, and 6 hours is typical. The battery does not require any maintenance, and has an expected lifetime of 2.5 years or 300 battery discharge cycles, whichever comes first.



Battery Charger

Quickly recharge the Powerheart AED G3 Pro Rechargeable Battery to its full capacity in just three hours (4.5 hours if rechargeable battery is completely depleted). It also accepts all standard IEC main power cables.



168-6000-001 Carry Case

9044-001

Carrying Case for G3 AEDs includes storage pouch on back for additional supplies, and cutout window for easy viewing of pad expiration dates.

## **S**PECIFICATIONS

#### **DEFIBRILLATOR**

Operation Semi-automatic and manual
Waveform Biphasic truncated exponential
Energy (J) range available Escalating Variable Energy (VE) 105J to
360J
Protocols 5 energy protocols available; Aligned with

new AHA/ERC 2005 Guidelines protocols

Factory default (Nominal) 200VE, second and subsequent shocks at

300 VE

Control buttons Shock button and manual override
Voice prompts Comprehensive voice instructions guide

user through rescue process

Display content Displays written instructions to guide user through rescue process, SmartGauge

Battery Status Indicator, Service Indicator, Pads Indicator, Text Display, ECG Display 8.9 cm (3.5") diagonal transreflective TFT display with 320 x 240 pixels (quarter

VGA). Resolution is 4.47 dots/mm

(113.5 dots/in.)

Visible indicators RescueReady Status Indicator
Audible alerts Voice Prompt, System Alert

Synchronized shock Built in automatic synchronization feature

Pacemaker pulse detection Ye

Display specifications

Programmable Yes, via MDLink® software (included)

Pediatric capability Yes
Warranty 7 years

### **PADS**

Minimum combined surface area 228cm<sup>2</sup> Extended length of lead wire 1.3m

Supplied Pre-connected to the AED

Type Adult, pre-gelled, self-adhesive,
disposable, polarized defibrillation pads

Shelf life 2 years

### **BATTERY**

Type IntelliSense lithium battery

Warranty 1 year or 12 hours of use, whichever

occurs first

Type Rechargeable Battery

Warranty 1 year

#### **OPTIONS**

	ECG Patient Monitoring Kit IR Communications cable	Includes 3-lead monitoring cable Wireless IR data transfer
AUT	OMATED SELF-TESTS	
	Daily	Battery, pads (presence and function), internal electronics, SHOCK button, and software
	Weekly	Battery, pads (presence and function), internal electronics, partial energy charge, SHOCK button, and software
	Monthly	Battery, pads (presence and function), internal

#### EVENT DOCUMENTATION

ENT DOCUMENTATION				
Internal memory	60 minutes ECG data with event annotation			
Multiple rescue	Yes			
Playback	Viewable via RescueLink® software via PC			
Communications	Infrared Data Transfer for PC			
	with Windows			
Clock synchronization	Precise synchronization of AED time			
	to dispatch time			

and software

electronics, full energy charge, SHOCK button,

#### PHYSICAL DIMENSIONS

Height	8 cm (3.3 in)
Width	27 cm (10.6 in)
Depth	31 cm (12.4 in)
Weight	3.1 kg (6.9 lb)

## **Order Information**

MODEL	DESCRIPTION
9300P-501	Powerheart AED G3 Pro Automated External Defibrillator

Each AED package includes: (1) defibrillator, (1) IntelliSense battery (9145), (1) pair of defibrillation pads, and (1) Quick Start Tool Kit: includes Quick Start Guide, CD-ROM with AED Manual, Training Video, RescueLink and MDLink.

9300P-601 Powerheart AED G3 Pro Automatic External Defibrillator with Rechargeable Battery. Battery charger sold separately.

Each AED package includes: (1) defibrillator, (1) Rechargeable Battery (9144), (1) pair of defibrillation pads, and (1) Quick Start Tool Kit: includes Quick Start Guide, CD-ROM with AED Manual, Training Video, RescueLink and MDLink.

### For more information contact Cardiac Science at:

Corporate Headquarters 3303 Monte Villa Parkway Bothell, WA 98021 USA Phone: +1.425.402.2000 Toll Free US: +1.800.426.0337

Manufacturing Operations 500 Burdick Parkway Deerfield, WI 52521, USA Phone: +1.608.764.1919 Toll Free: +1.800.777.1777 US/Canada

Customer Service/Order Entry Phone: +1.425.402.2690 Toll Free: +1.800.991.5465 Fax: +1.425.402.2010

Email: customerservice@cardiacscience.com

US/Canada Technical Support Phone: +1.425.402.2690 Toll Free: +1.800.991.5465 Fax: +1.425.402.2022

Email: techsupport@cardiacscience.com

Nasdaq: CSCX www.cardiacscience.com



# CARDIAC SCIENCE

International Operations

Kirke Vaerloesevej 14, DK-3500, Vaerlose Denmark Phone: +45.4438.0500 Fax: +45.4438.0501

United Kingdom Operations Cardiac Science Holdings Limited Unit 13, Crown Industrial Estate, Canal Road, Temperely Altrincham, Cheshire, WA14 1TF, United Kingdom Phone: +44.161.926.000 Fax: +44.161.926.0002

International Customer Service/Tech Support Contact Local Representative

